# Instructions For Use

B-CURE<sup>®</sup> LASER  
SPORT PRO

GOOD  
ENERGIES<sup>®</sup>

## Content

English

Instructions for use.....03

Suomi

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Svenska

Bruksanvisning .....24

Dansk

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For more convenient and efficient treatment, it is recommended to use the B-Cure Laser adjustable stand (not included).

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
## Content:

1. Component legend
2. Battery charging
3. Battery warnings
4. Safety precautions
5. Contraindications
6. Operating instructions
7. Treatment protocol
8. Control panel and troubleshooting
9. Technical specifications
10. Device maintenance
11. Warranty

### 1. Component legend

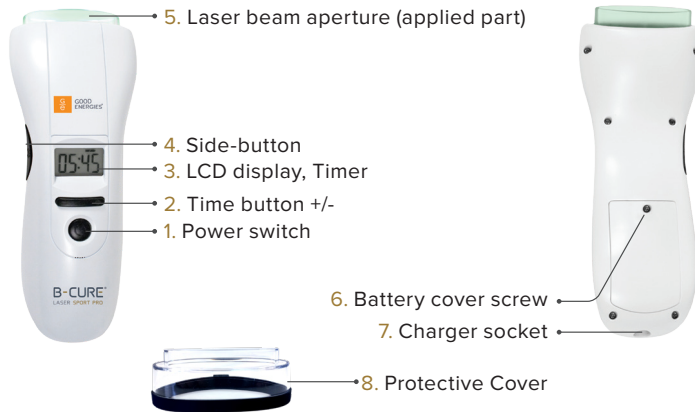
#### Package content:

- B-Cure Laser device
  - B-Cure case
  - Instructions For Use
  - Batteries
  - Charger (please see battery charger technical specs. on page 10)
  - Screwdriver for battery cover
- Upon opening the package, please verify that it contains all the above mentioned equipment. Please make sure the device is unharmed. If anything is missing or damaged, please contact the distributor immediately after purchasing.

### 2. Battery charging

3pcs of AAA batteries are included. Install the batteries and fully charge them by plugging the charger into an electrical outlet and connect the charger to the Charger socket (#7). Blinking dots shown on LCD display (#3) indicates batteries are being charged. Battery charge time is approximately 7 hours, or until the word "FULL" appears on the screen. We recommend you charge the device once every three days, or when the device indicates low battery level. The device cannot be used during the charging process.

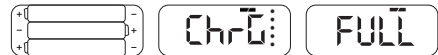### 3. Battery warnings

The battery charger cable may present a choking or trip hazard if the device is left unattended in the vicinity of infants and small children. Recharge the device battery in a safe place, away from the reach of children. Store the battery charger in a safe place when not in use.

- The AC/DC charger that comes with the device is the only charger allowed for use.
- Do not take apart the batteries, heat them over 100°C or burn them.
- Discard the batteries at an appropriate recycling center.
- Do not turn the device on while it is being charged.
- Do not, under any circumstances, charge ordinary, non-rechargeable batteries; it might cause a fire.

### 4. Safety precautions

-  • The patient is the intended device operator. The patient may safely use the device including all its functions, e.g.: charging and changing the batteries.
- Use the batteries recommended in this instruction of use.
  - **Never aim the laser beam at the eyes.**
  - It is always recommended to consult your doctor. Or using the device under doctor's instruction.
  - If you are pregnant do not use the device to treat the stomach.
  - Do not treat an area with a malignant tumor.
  - When charging battery, position the device in such a way that will facilitate access to the plug.
  - If the performance level of one of the device components (laser energy, charger, etc.) changes, please contact the distributor.
  - Warning: If you are allergic to polyester, nylon or plastic, please don't use this device.
  - Warning: Keep the device out of reach of infants, young children or pets to avoid hazards.
  - Warning: No modification of this device is allowed.
  - Warning: If this device has been modified, appropriate inspection and testing must be conducted to ensure it is safe to use.
  - Warning: Avoid eye exposure when operating the device, otherwise may lead to eye damage or blindness.
  - Warning: Removal of rechargeable batteries from its compartment is not recommended. Using other charging adaptors may lead to battery damage, leakage or explosion.
  - Warning: If the device will not be used for a long period, it is recommended to remove the batteries (e.g. alkaline batteries) from its compartment to avoid leakage, which may cause damage to the device. Do not store the device with alkaline batteries installed to avoid battery leakage damage.
  - Warning: Do Not dry the device by using fire, microwave machine or other heating sources which may cause damage to the device or lead to battery explosion.

- Precaution: Do Not clean the device with hot water or steam, which may cause damage to the device and lead to malfunction.
- Precaution: Do Not use the device near magnetic field, which may cause potential interference or display malfunction.
- Suggestion: The user should keep the device in a static state when using it, or the device is not allowed to be used in traffic and on-board conditions.

## 5. Contraindications

None

## 6. Operating instructions

### Protective Cover (#8)

Protective cover is designed for hygienic use of the device. Cover is to be used when treating open wounds, contagious conditions, dirty areas or for multiple people. Before each use of the Protective Cover (#8), check its condition is intact and free from damage. Clean and Sterilize it before and after each use. After checking and sterilizing, you can simply attach the Protective Cover (#8) gently to the device. The silicone seal around the Protective Cover (#8) will keep it attached to the device.

### 6.1 Switch the device ON

Press the Power switch (#1). After a short beep the display will switch on and a timer will be displayed (#3). In case the device was stored in extreme temperatures (lower than 0°C or above 40°C), allow the device to rest for approx. 20 minutes until the following use.

### 6.2 Set treatment time

Press Time button +/- (#2): on its right side to increase time (+) or its left side to decrease time (-). The timer is adjustable in increments of 30 seconds.

### 6.3 Laser beam activation method

**After switching ON the device and setting treatment time, activate the laser beam.**

**Step 1:** Press the Side-button (#4) to activate Laser beam aperture. Green light is on and laser beam is activated

**Step 2:** While pressing the Side-button (#4), use your another finger to press the Power button simultaneously.

**Step 3:** Release both buttons. Now the laser beam remains active and treatment can begin. Place the device gently in full contact with the skin. Do not treat on top of clothes or bandaging. To stop Stay-on mode, press the Sidebutton (#4) again.

### Stay-On mode demonstration after setting treatment time

Step 1  
After setting the treatment time, press the Side-button #4

Step 2  
While pressing the Side-button #4, use your another finger to press the Power switch simultaneously.

Step 3  
Release both buttons. Green light remains on and the laser beam is activated.

Place the device gently in full contact with the skin. Do not treat on top of clothes or bandaging. During treatment, you will see a green light indicating the device is functioning. “Beep” sounds at 3 second intervals indicate that the device is functioning and that the stopwatch is counting down until reaching “00:00” at which time the device’s functioning will automatically end and the “Beep” sound will stop. Hold the device on the skin steadily and motionless during the whole duration of the treatment.

### 6.4 End of treatment, reset and turn OFF

To switch off the device, press the Power button once, followed by a beep sound. Release the button and the display will go blank. If the planned treatment time has not been fully utilized, the timer will display the remaining time at startup. To reset the timer, switch off the device, then press the Power switch (#1) for longer than 2 seconds while switching on the device. To turn off the laser during treatment, press first the side-button (#4), then press the Power button to turn off the device as normal.

## 7. Treatment protocol

### 7.1 Intended use


B-Cure Laser is intended for the management of pain including: Chronic back pain; Chronic neck pain; Knee pain; Carpal Tunnel Syndrome (CTS); Fibromyalgia; Myofascial Pain Syndrome (MPS); Temporomandibular Joint Disorder; May assist as an adjunctive treatment in treating acute and chronic wounds, Acupuncture treatment.

## 7.2 Treatment course and dosage

Start treatment 1,5 min, Treat twice a day. Increase the treatment time with 1 minute daily until you reach a treatment time of 6 minutes. If you experience increased pain following treatment, stop the treatment for 24 hours then start again with a reduced dosage. You may increase the dosage to 4 treatments a day. In cases of acute pain, trauma or non-chronic aches, you may apply a dosage of 6 minutes from the first treatment.

## 7.3 Coverage

The laser beam size is 4.5 cm long and 1 cm wide. If the area that requires treatment is larger than the 4.5/1 cm, repeat treatments on the adjacent area until you've covered the whole area.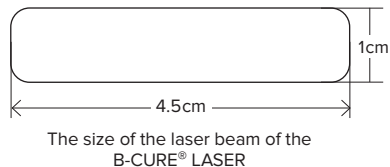

## 7.4 Treatment recommendations for common conditions

**The recommended time for each point of treatment is 6 minutes, 2-4 times a day.**

For all recommended treatment therapies, consult the human body illustration, page 8.

- 1. Chronic back pain** - Treat lengthwise along the painful vertebrae of the lower back, to the left and the right of the spine. For Ischia - pain radiating to the lower limb - Identify the focal point of the pain by touch and treat it.
- 2. Chronic neck pain** - Treat **a)** Lengthwise, along the vertebra. **b)** Treat each side of the neck, lengthwise; under the ear lobe, and exactly at the indentation behind the jaw.
- 3. Knee pain** - Treat the sore knee **a)** On the back of the knee along the inner crease. **b)** On the outer side of the knee, lengthwise. **c)** On the inner side of the knee, lengthwise. You can also treat the front of the knee, under the kneecap.
- 4. Carpal Tunnel Syndrome (CTS)** - Treat the sore wrist **a)** On the inside, along the width of the joint - on the crease. **b)** On the inside, perpendicular to 'a', lengthwise. **c)** Treat the length of the thumb, along the pad. You can also treat the back of the hand, at the gap between the thumb and the bone of the index finger towards the joint, and from both sides of the joint, lengthwise.
- 5. Fibromyalgia** - Treat the aching area.
- 6. Myofascial Pain Syndrome (MPS)** - Treat the aching area.
- 7. Temporomandibular Joint Disorder** - Treat along the length and width of the TemporoMandibular joint.
- 8. Adjunctive treatment for acute and chronic wounds** - Treat over the entire wound area.


**Tip:** Always start from the most painful point and then move to other spots, so as to ensure you are treating the central problem first, to the best of your knowledge.

**For wound treatment**

Set treatment time to 6 minutes. Activate the laser beam and hold the device about 1 cm above the wound, motionless, until the termination of the set treatment time. If the area of the wound exceeds the size of the laser beam area ( 1 x 4.5 cm), set the stopwatch again to 6 minutes and treat the adjacent treatment point and repeat until the whole surface area of the wound has been covered.

In order to ease the holding of the device and improve posture, it is recommended that the B-Cure stand should be used.


Using B-Cure Laser Pro to treat diabetic wounds and hard to heal wounds, is in addition to the standard treatment and does not replace it. The treatment protocol for diabetic and hard to heal wounds; 2 minutes on the margins of the wound and 30 seconds on the bottom, throughout the whole of the wound. Make sure, to perfectly cover the margins of the wound measuring 3 millimeter on each side.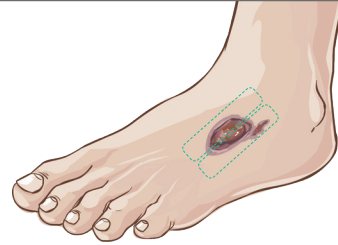





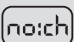


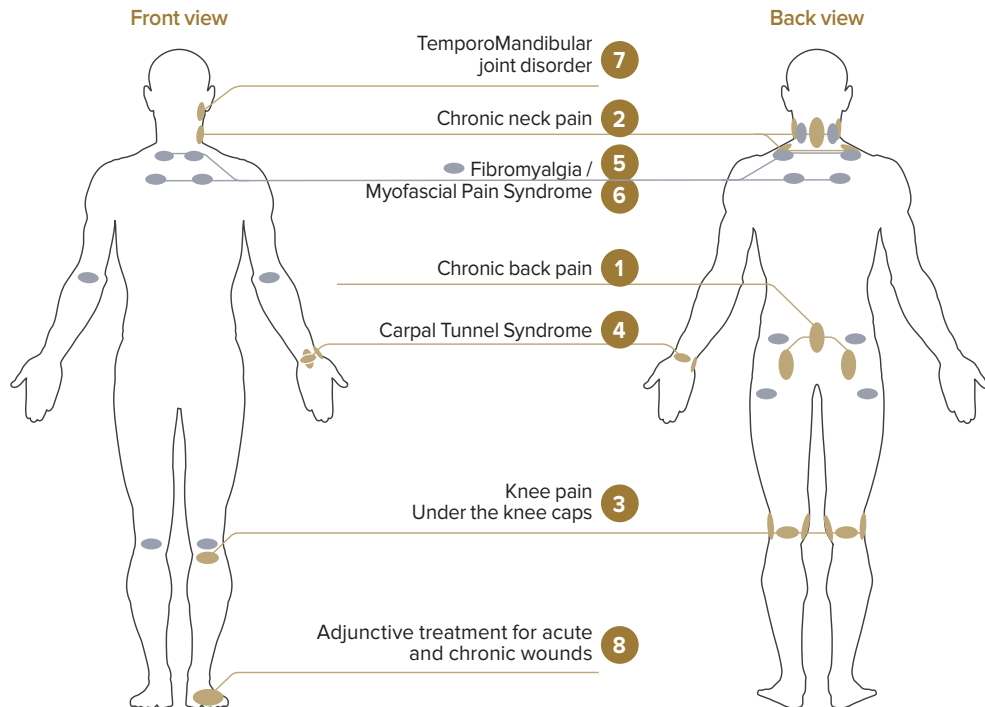
B-Cure adjustable Stand

## Human body illustration with recommended treatment points:

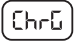- Note the illustrated beam direction
- You can treat any point in the body except the eyes
- In case of a raw wound or skin tissue, do not place the device in full contact with the skin. In any other case, place the device on the skin itself.
- Do not treat on top of clothes or bandaging.
- Do not move the device during treatment.
- It is advisable to treat all the recommended points, to ensure treatment efficiency.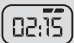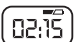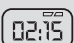
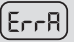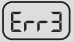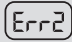
## 8. Control panel and troubleshooting

Display	Status
	Batteries charging
	Batteries fully charged
	The icon- * is displayed to indicate the device is currently operating
	Full battery power
	Low battery power, two beeps will sound after startup as a reminder to recharge. Charge the batteries
	No battery power remains. Charge the batteries immediately
	The device will shut down automatically after 10 beeps due to lack of battery power. Charge the batteries
	Batteries are not charging. Switch the device off then on again to fix this error. If the problem persists, switch off the device, remove the batteries and then reinsert them. If the problem persists, ship or deliver personally the device to the service center.
	The indication "LOW" is displayed when laser power level is below the required level. Treatment should be stopped. The cause may be one of the following: 1. Battery power is too low. 2. Diode lifespan has reached its end and should be changed at the service center. 3. The device is operated in a different environment than for which it was designed, e.g. temperatures higher than 40°C. Charge the batteries, switch on the device and see if the writing "LOW" has disappeared. If not, verify that room temperature is not exceptionally high, and if necessary move away from the hot area. If not higher than 40°C and the writing "LOW" is still displayed - ship the device to the service center for diode change.
	Additional signals for errors of this type:       Hardware/software error has occurred. Switch the device off, then on again, to fix this error. If the problem persists, switch off the device, remove the batteries and then reinsert them. If the problem persists, ship or deliver personally the device to the service center.

## 9. Technical specifications

Property	Units	Specifications, value	Description
Laser type		GaAIAs	Solid state laser diode
Maximum power	mW	250	Max
Wave length (infrared)	nM	808	
Pulse frequency	kHz	10.9	
Laser pulse duration	µs	31.5	
Diode lifespan	hours	1000	
Energy per minute	Joul	5	
Laser beam dimensions (LXH)	mm	Length 45 x width 10	at opening
Beam divergence	degrees	±3	
Green control light indicator	nM	520	Green light beam signaling device is operating
Timer setting	minutes	00:00 to 99:30	
Timer intervals	seconds	30	increment or decrement
LCD display dimensions	mm	26 x 15	Visible area
Timer display		4 digits (88:88)	1-second intervals
Beep warning		yes	Display: Device on, off, operating, low laser power, errors
Power supply	W	3.2	maximum
Automatic shutdown		2 options	Automatic shutdown when not in use for 10 minutes; 5 minutes when timer is set to 0 (00:00)
Battery type	V	AAA850mAh 1.2V	rechargeable batteries (NiMH) (Battery Life: 500cycles)
Battery working duration	hours	2	with fully charged batteries
Battery charging time	hours	5-7	with discharged batteries
Batteries charging current	mA	300	minimum

## 9. Technical specifications - continued


Property	Units	Specifications, value	Description
Power supply 1	VAC	100 - 240VAC	Dongguan Shilong Fuhua Electronic Co., Ltd./ EU plug: UES06WV-090060SPA. US/JP plug: UES06WU-090060SPA. UK plug: UES06WB-090060SPA SWITCHING POWER ADAPTOR INPUT: 100-240VAC, 50/60Hz, 0.2A OUTPUT:9.0VDC, 0.6A, 5.4W
Power supply 2	VAC	100 - 240VAC	JDI Electronics Factory, Model: HWU18A-090M INPUT: 100 -240VAC 50/60Hz 0.5A-0.2A OUTPUT: 9.0VDC 0.6A, 5.4W
Dimensions	mm	200 x 70 x 40	
Weight	grams	175	
Operating conditions	°C, %	Temp 0°C to 40°C Humidity 15% to 90% Atmospheric pressure: 70Kpa to 106Kpa	without condensation
Transport and Storage conditions	°C, %	Temp -25°C--70°C Humidity 10% to 95% Atmospheric pressure 50Kpa to 106Kpa	without condensation

## 10. Device maintenance

1. Clean the laser beam aperture (#5), using disinfective solution. Clean between sessions if needed and before handing it to another person's usage.
2. While cleaning, be sure to prevent wetting the lens.
3. Keep the device in proper storage and work conditions as specified in the technical specification table, do not leave the device in the car in hot weather. When not in use, store the device in a dry room and protect it against extreme moisture, heat, lint, dust and direct sunlight.
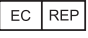
## 11. Warranty

The warranty is for 24 months according to company policy, as is detailed on warranty certificate. If batteries and/or a charger are supplied with the device, they are not included in the warranty. The manufacturer/ distributor is responsible for performance, safety and reliability of the equipment only on condition that all repairs are carried out exclusively by company-authorized personnel.

Consult accompanying documents 	Atmospheric pressure limitation 	Humidity limitation 
Electrical and electronic equipment 	Temperature limitation 	Caution, consult accompanying documents 
Type BF  Recycling 	IP22 – International Protection	Manufacturer 
3 x 1.2V AAA Rechargeable batteries   9V, 300 mA (For charging via AC/ DC adaptor)		CE marking of conformity 
Authorized representative in the European community 	Stacking limit by number 	
 This way up 	Fragile; handle with care 	Keep away from rain 

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**Manufacturer: Biocare Enterprise Limited**
**Distributor: Erika B-Cure Laser Ltd.**
**Maahantuoja: Terveystekniikka Valko Oy**